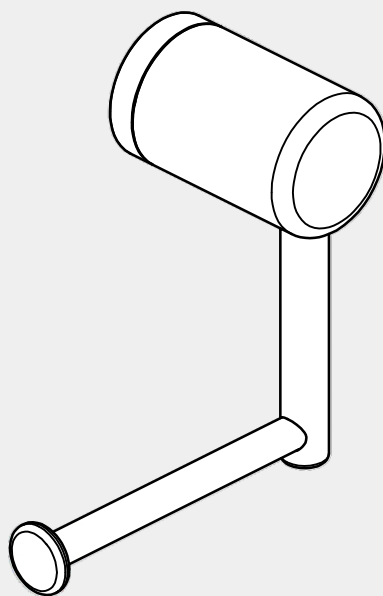
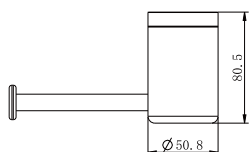
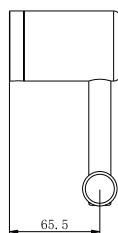
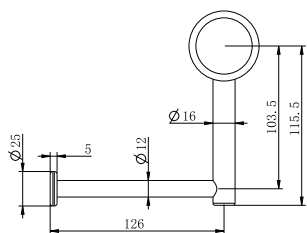


# Mecca Care Heavy Duty Toilet Roll Holder

# Nero



## Drawing



## Specifications

**SKU**  
NR CR3286

### Warning:

These rails are designed to be mounted onto wall studs or masonry walls. Installation is recommended to be completed by qualified trades person who complies with the AS 1428.1-2009 design for access and mobility standard. It is recommended rail location is determined by a qualified occupational therapist or trained professional to the AS 1428.1-2009 standard. Correct installation is required to ensure safe, long term operation. Incorrect mounting can result in reduced capacity of the rail which can result in injury to user and damage to the wall.


## Finish


 **Chrome**  
NR CR3286CH

 **Gun Metal**  
NR CR3286GM

 **Brushed Bronze**  
NR CR3286BZ

 **Matte Black**  
NR CR3286MB

 **Brushed Gold**  
NR CR3286BG

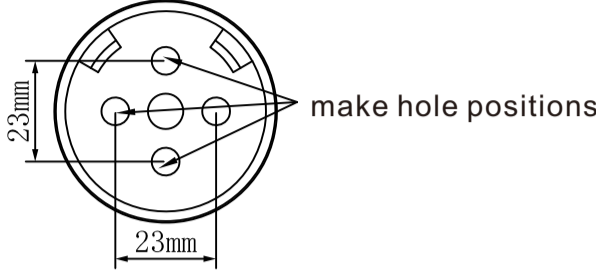
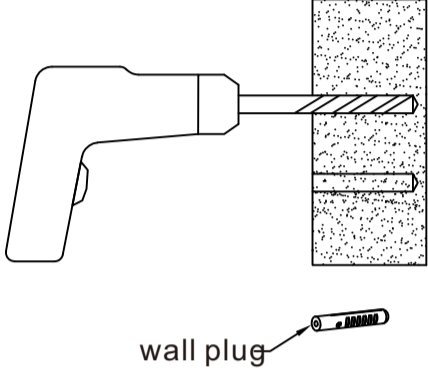
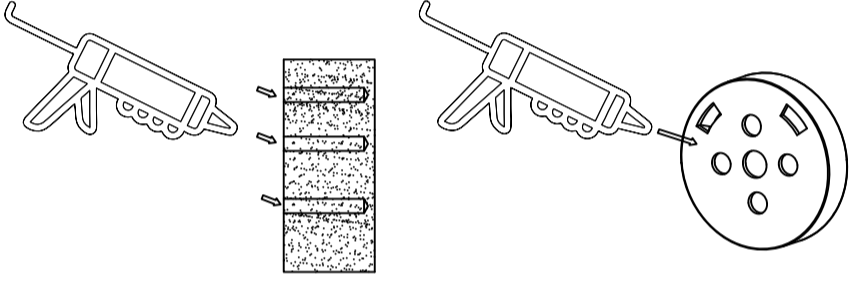
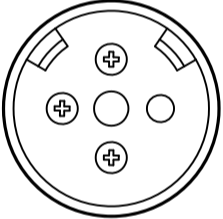
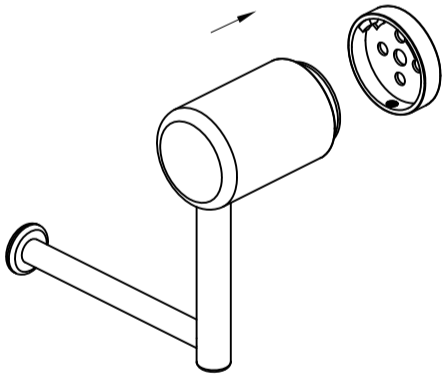
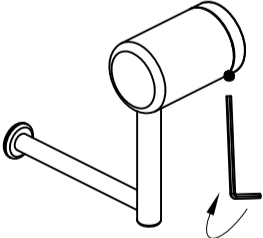
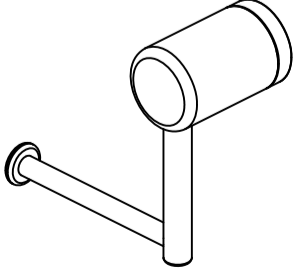
 **Brushed Nickel**  
NR CR3286BN

## Features

1. Australian owned and designed.
2. Provides a helping hand and a handy toilet roll holder.
3. The direction can be changed from left to right during installation.
4. 150 kg capacity when installed into a supported wall.
5. Gun metal, brushed gold and brushed bronze colours are PVD coated for long term durability.

\*Dimensions are nominal measurements only.

# Installation Instruction

	<p>1. Locate wall studs, install into at least one stud. Place wall seal/mounting bracket on wall at correct height over centre of stud and mark hole positions on wall, at least 3 holes. (3 supplied screws for bracket).</p>
	<p>2. Drill pilot holes into wall stud approximately 60mm deep to suit mounting screws (and wall plugs if required).</p> <p>Tip: 3mm hole for screws into timber or 8mm hole for wall plugs.</p>
	<p>3. Apply small amount of silicon to mounting bracket and wall in and around the screw holes.</p>
	<p>4. Install mounting brackets by screwing onto wall.</p> <p>Tip: Use appropriate mounting screws for the wall stud material, supplied screws are for timber wall studs.</p>
	<p>5. Push roll holder onto wall brackets, refer to left picture.</p> <p>Tip: The direction can be changed from left to right during installation.</p>
	<p>6. Tighten grub screws with allen key.</p>
	<p>7. Finished.</p>

# Nero

info@nerotapware.com.au  
nerotapware.com.au

11-13 Buontempo Road  
Carrum Downs Vic 3201



**MECCA CARE  
DDA CARE 32MM**